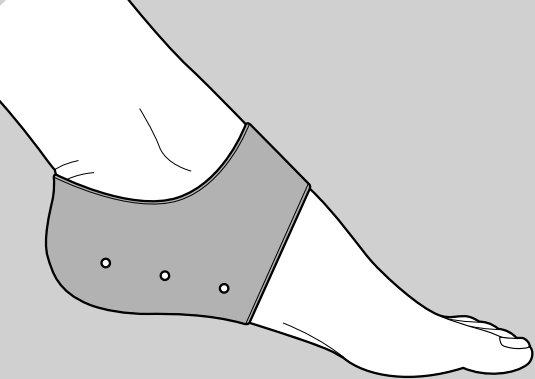
# Fersenschutzhauben

## Heel protection covers

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 2027-07-01

 PS Product Services GmbH,  
Alte Heerstraße 6, 53757 St. Augustin, Germany  
info@product-services.de

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**en**

#### Dear Customer

 Read these instructions carefully and keep them for future reference. If you give this product to another person, remember to also include these instructions.

#### Intended use/indication

The heel protection covers relieve pressure on the heels - e.g. in the case of heel spurs, reduce pressure and pain, and protect your feet from mechanical friction. They may be used for their intended purpose only and are designed for personal use only. **Consult your doctor if you have any questions.**

The heel protection covers are made of skin-friendly thermoplastic elastomer – a particularly soft material that provides excellent padding. The perforated structure allows for effective air circulation. The covers, one for your left foot, one for your right, are suitable for all standard shoe sizes and can easily be worn under socks/tights. The slip-on shape of the heel protection covers prevents them from slipping while they are being worn. The heel protection covers are washable by hand and reusable.

#### Contraindications

If you suffer from diabetes or from circulatory or skin disorders, use the product only after consulting your doctor.

#### Safety warnings

- Keep the product and packaging materials out of the reach of infants. They pose several risks, including the risk of suffocation!
- If you experience redness, swelling or persistent discomfort when wearing the product, remove it immediately. It must not restrict the flow of blood. The product must not be worn while asleep or by anyone who is unable to indicate discomfort.
- Use the product only on unbroken, healthy skin. The skin must be free of oils, creams, ointments, etc.
- Serious incidents which occur while wearing the product must be reported to the manufacturer and the relevant authorities.
- **The product must be applied/positioned as shown in the illustration (at the top of these instructions).** When putting them on, pay attention to the alignment of the holes - the 3 pairs of holes are next to each other, not on top of each other.

#### Care and storage

- Wash the product under running water before using it for the first time and after each subsequent use. Allow to dry at room temperature. Do not place on a radiator to dry!
- In the course of normal use, the product releases very small amounts of oil. You may apply baby powder to the product, as required.
- Store the product in such a way that it is protected from dust, moisture, direct sunlight and extreme temperatures. Ensure that it is also protected from sharp or pointed objects.

#### Disposal

- Dispose of the product and packaging in accordance with current environmental regulations.

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